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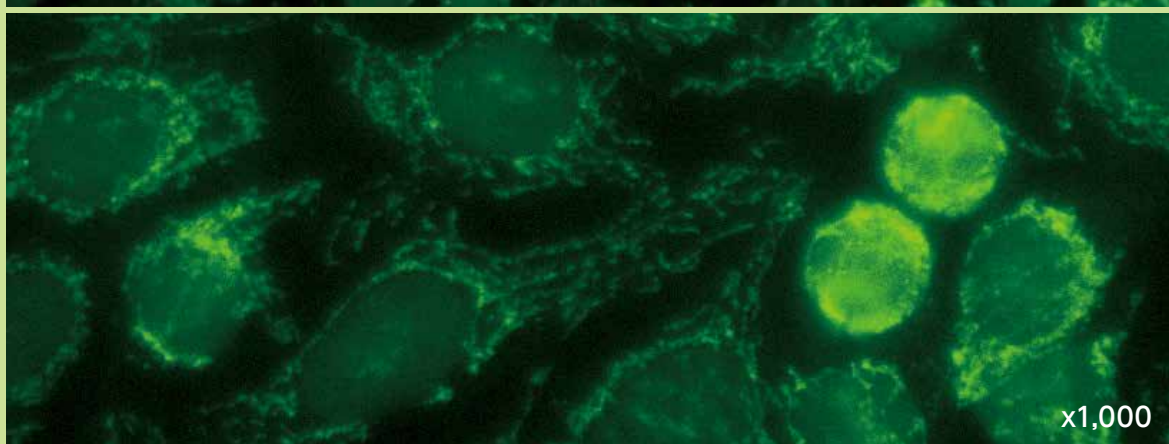
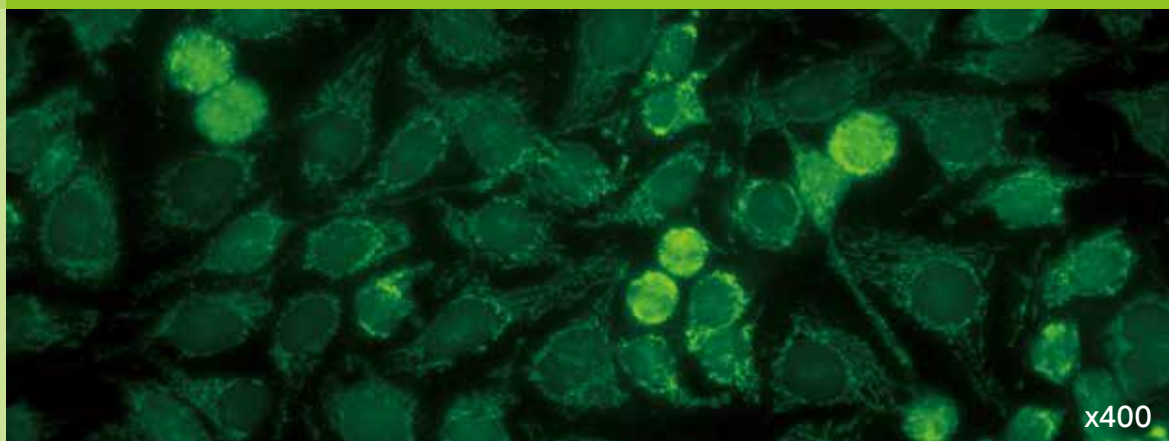
1. 森藤 隆夫, 黒田 聖仁. 抗ミトコンドリア抗体(2). MBL自己免疫レポート. 1990;9:1-9.
2. Nishio A, Van de Water J, Leung PS, Joplin R, Neuberger JM, Lake J, et al. Comparative studies of antimitochondrial autoantibodies in sera and bile in primary biliary cirrhosis. *Hepatology*. 1997;25(5):1085-9.
3. Nardi N, Brito-Zeron P, Ramos-Casals M, Aguilo S, Cervera R, Ingelmo M, et al. Circulating auto-antibodies against nuclear and non-nuclear antigens in primary Sjogren's syndrome: prevalence and clinical significance in 335 patients. *Clin Rheumatol*. 2006;25(3):341-6.
4. Skare TL, Nisihara RM, Haider O, Azevedo PM, Utiyama SR. Liver autoantibodies in patients with scleroderma. *Clin Rheumatol*. 2011;30(1):129-32.
5. Chan EK, Damoiseaux J, Carballo OG, Conrad K, de Melo Cruvinel W, Franciscantonio PL, et al. Report of the First International Consensus on Standardized Nomenclature of Antinuclear Antibody HEp-2 Cell Patterns 2014-2015. *Front Immunol*. 2015;6:412.
6. European Association for the Study of the Liver. Electronic address eee, European Association for the Study of the L. EASL Clinical Practice Guidelines: The diagnosis and management of patients with primary biliary cholangitis. *J Hepatol*. 2017;67(1):145-72.
7. Shuai Z, Wang J, Badamagunta M, Choi J, Yang G, Zhang W, et al. The fingerprint of antimitochondrial antibodies and the etiology of primary biliary cholangitis. *Hepatology*. 2017;65(5):1670-82.
8. Zheng B, Vincent C, Fritzler MJ, Senecal JL, Koenig M, Joyal F. Prevalence of Systemic Sclerosis in Primary Biliary Cholangitis Using the New ACR/EULAR Classification Criteria. *J Rheumatol*. 2017;44(1):33-9.
9. Damoiseaux J, Andrade LEC, Carballo OG, Conrad K, Franciscantonio PLC, Fritzler MJ, et al. Clinical relevance of HEp-2 indirect immunofluorescent patterns: the International Consensus on ANA patterns (ICAP) perspective. *Ann Rheum Dis*. 2019;78(7):879-89.